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ELECTRICAL & CONNECTIONS

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The EFT-3002 uses 12 volts DC and comes from the manufacturer with a power supply using a North American standard power cord. Adapters for other countries can be used. The power supply is able to operate from mains voltages of between 100 and 240 volts 50–60Hz AC.

Battery operation is also possible. Two power input connectors are available. Either connector can be used for operation or both at once for fail-safe operation in case of power failure from one power source. A car-to-freezer adaptor cord is included.

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HEAT TRANSFER MEDIA

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The EFT-3002 is designed to use ethanol, isopropyl alcohol (available in most pharmacies and supermarkets), or ethylene glycol as the freezing chamber heat transfer media. Isopropyl alcohol with a purity of 90% is best. During exposure to air the alcohol will absorb moisture. This contamination will cause the alcohol to become thick at low temperatures. Do not reuse the alcohol if ice crystals have formed on the surface.

The use of other heat transfer liquids is not recommended. Fluids such as acetone, methanol, and isopentane should be avoided because of their caustic effects on plastic parts contained in the freezer. The use of non-approved fluids will void all warranty from the manufacturer and dealer. Do not transport the unit with alcohol in the freezing chamber.

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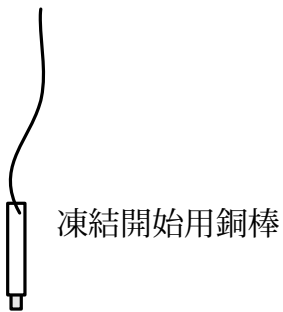
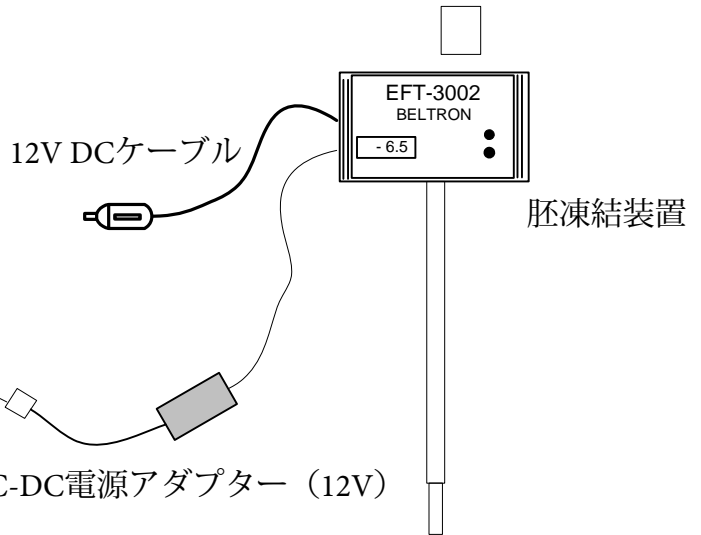
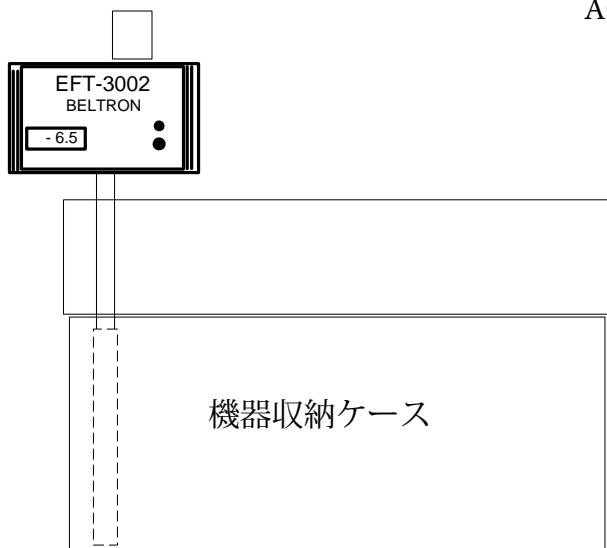
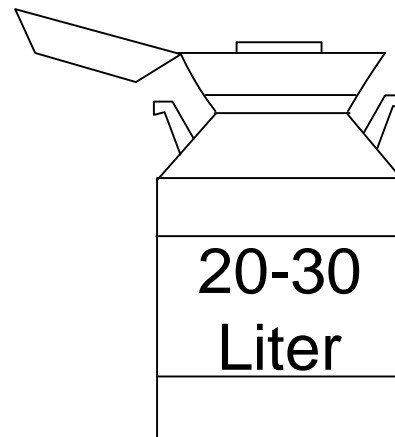
SPECIFICATIONS

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Mains Voltage Adapter: 100–240 Volts A.C. 50–60 Hertz
DC Input: Two available, 12 volts DC @ 2.5 Amps max., 0.027 Amp min.
Fuse: 3.5 Amp self-resetting (not user replaceable)
Weight: Freezer 4.2 lb / 1.9 kg; Case 9 lb / 4.1 kg; Shipping 19 lb / 8.6 kg
Dimensions:
• Freezer body: 6.5 in L x 9.5 in W x 3.75 in D (16.5 x 24 x 9.5 cm)
• Freezer rod: 17 in L x 1 in diameter (44 x 2.5 cm)
• Case: 28 in L x 12.75 in W x 12 in D (73.7 x 32.4 x 30.5 cm)
Shipping dimensions: 29 x 14 x 13 in (73.6 x 35.6 x 33 cm)
Straw capacity: 32 x 0.5 ml (one straw per hole) or 96 x 0.25 ml (three straws per hole)
Ramp rate: 0.5°C per minute (mechanically limited to approx. 0.8°C per minute with no power)
Environmental limits: -50°C to +50°C; 0 to 95% humidity, non-condensing
Start temperature: -6.5°C ±0.05°C
End temperature: approx. -35°C
Safety compliance: CE, UL, GS
Included accessories: 120/240 power supply, copper seeding rod, alcohol bottle, 12-volt auto connector, 12-volt battery clamp adapter, freezing chamber cap

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WARRANTY SERVICE & REPAIR

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This product carries a 5-year limited warranty on parts and materials. Beltron Instruments warrants this equipment to be free from defects in material and workmanship when used under normal conditions and according to the operating instructions.

For a period of 5 years from date of delivery, any part found to be defective will be repaired or replaced, free of charge, at the discretion of Beltron Instruments. All repairs are to be performed by Beltron Instruments or its authorized service representatives.

All shipping charges to Beltron and back to the customer must be prepaid by the customer. Any damage resulting from misuse or use of unapproved fluids will void this warranty. The above warranty is the full and only warranty by the manufacturer.

For warranty service or repair, send to your Beltron representative or directly to:
Beltron Instruments
1145 Rodriguez Ct.
Longmont, CO 80501-3828 USA
1-979-574-6956
sales@beltroninst.com
www.beltroninst.com

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EFT-3002

OPERATING INSTRUCTIONS

When you first receive the freezer run through a mock freezing program checking the start temperature and freezing rate.

1. Slowly insert freezer into liquid nitrogen "LN₂" tank

Caution: Direct contact with liquid nitrogen can cause severe damage to skin.


2. Attach the power cord. It can be powered from 110v, 220v with power supply or 12v from battery. Any combination of two power sources can be attached simultaneously as a backup.

Tip: The freezer can also be operated in a vehicle using the 12V power.

Freezing chamber temperature will be displayed.

3. Fill freezing chamber with 70% or better isopropyl alcohol, to between the holes and top edge of freezing chamber. (about half a chamber cap full)

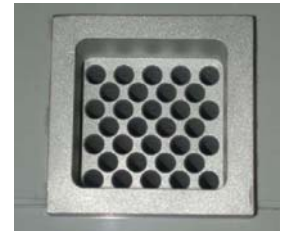Tip: Using alcohol that has been kept in the freezer will reduce the cool down time and save liquid nitrogen.

4. Place cap on freezing chamber. The controller will automatically allow the freezing chamber to cool down to the start temperature (-6.5C). This will take approximately 45 minutes from room temperature. Once the temperature has stabilized for 1 minute you may place the prepared embryos into the freezer with the straw ID extending above the chamber.

Tip: Check the display ramp indicator to verify that the start button has not been pressed. If it has, just disconnect power and the freezer will reset to the start temperature.

5. Place the embryo straws into the freezing chamber.

6. Seed each of the straws by touching with the frozen copper rod.

7. Start the freezing process by pressing the red button. NOTE: Do Not depress the start button until all of the embryo straws are in the chamber. Ramping can be stopped by pressing the start button again. The temperature will then ramp down according to the selected ramp rate until it has dropped to the finish temperature (colder than -30 degrees C). The freezer can be removed from the LN₂ tank and placed in the PVC tube in the carrying case. The embryos are then ready to be transferred to liquid nitrogen storage.

Tip: If power is lost after this step DO Not Panic. Leave power off and allow to freeze for 45 minutes, most of your embryos should survive.

8. The embryos can now be transferred to a properly labeled cane that has been submersed in LN₂. Be sure to shake the excess alcohol off of the straws prior to plunging them into LN₂ to prevent the straws from freezing together.

After the straws have been removed, disconnect power from the freezer and place the protective cover over the chamber and invert the freezer to collect the alcohol. Or if you want to run another program just remove power from the freezer until display goes blank, then back on. In about 30 minutes the freezing chamber's temperature will have returned to its start temperature.